



Implant  
Direct™

simply smarter.



# Spectra-System® Prosthetic Catalog



ScrewPlant®

Locator® is a registered Trademark of Zest Anchor Company



# ScrewPlant and ScrewPlus Fixture-Mount Packaging

The implant is mounted on a color-coded fixture-mount, designed for use as a transfer and shortened for a straight abutment. A snap-on comfort cap is provided for the modified abutment. A cover screw is included in each vial. The ScrewPlant and ScrewPlus also include a 2mm extender retained by the cover screw for use as a healing collar.



Prepare surgical site with double cutting Drill



All of Implant Direct's implants come in a sterile vial



Each label is color-coded with two tear-off labels



Each cap is color coded by platform diameter of implant



Drop implant assembly out of vial onto a sterile surface



Remove implant assembly with a fixture-mount drill or Ratchet




Insert Implant into osteotomy



Index fixture-mount flat to labial



Remove fixture-mount

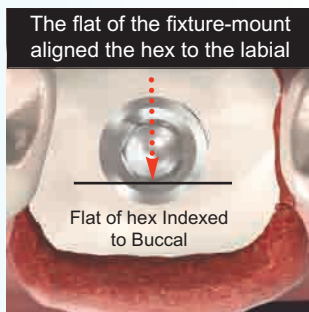

The flat of the fixture-mount aligned the hex to the labial

Flat of hex Indexed to Buccal



Remove the holder containing the cover screw & comfort cap



Snap-on comfort cap is located in the top portion of holder



Attach fixture-mount to corresponding color-coded implant analog



Remove colored section from fixture-mount through groove creating a final straight abutment



Use 30Ncm in hard bone. In soft bone, only use finger pressure



Check for occlusal clearance prior to attaching comfort cap



Test retention and rotate if necessary to increase retention



Suture soft tissue around comfort cap




## Orienting the 15° Angled Abutment

The flat side of the Fixture-Mount/Transfer [Fig. 1] and the angle of the 15° abutment [Fig. 2] are manufactured to align with the flat of the implant's internal hexagon.



Fig. 1  
Fixture Mount/Transfer

Fig. 2 The two-piece, 15° Angled Abutment consists of an angled body and a Fixation Screw

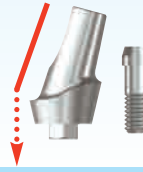

To establish the optimum angle of the abutment, index the flat surface of the Fixture-Mount/Transfer either toward or away from the desired direction of abutment angle at the time of implant placement.


### Option 1 : Attaching an Angled Abutment and **chairside procedures** for preparation



Remove the healing collar with a 1.25mmD hex tool



Attach the abutment with the 1.25mmD hex tool

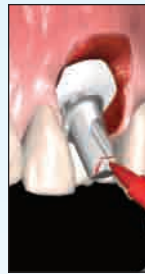

Mark the required modifications to achieve appropriate clearance



Detach the abutment with the 1.25mmD hex tool



Attach the abutment to the corresponding implant analog  
Reduce the abutment to achieve appropriate clearance

### Option 2 : Stage-One transfer followed by **laboratory procedures** for abutment preparation and provisionalization



During implant placement, index the flat of the fixture-mount




Block out the top of the fixture-mount and make a transfer impression




Attach the cover screw and submerge implant



Attach the fixture-mount to the implant analog and seat into the negative of the impression

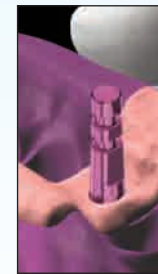

Pour the working cast in dental stone with soft tissue replication material

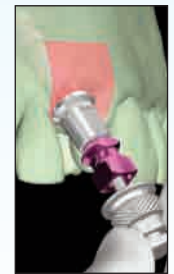


Remove the fixture-mount from the working cast with the 1.25mmD hex tool



Attach the angled abutment to the implant analog in the working cast with a 1.25mmD hex tool



Modify angle of abutment and refine margins as required

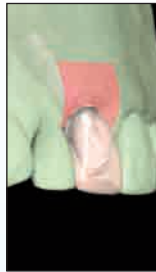

Fabricate the temporary crown



At stage-2 Surgery, attach the prepared abutment, tighten to 30 Ncm with a torque wrench



Occlude screw access hole and confirm contour of restoration



Cement the provisional restoration


### Common procedures for abutment placement, margin finessing, impressing and placement of restoration



Tighten to 30 Ncm with a torque wrench



Finesse the margins with a high-speed bur and copious irrigation prior to final impressing

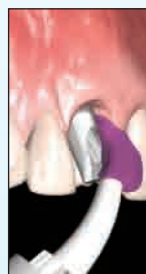


Make an impression following standard clinical procedures



Attach provisional restoration and cement onto the prepared abutment



Follow standard laboratory procedures, fabricate the final restoration and cement onto the prepared abutment

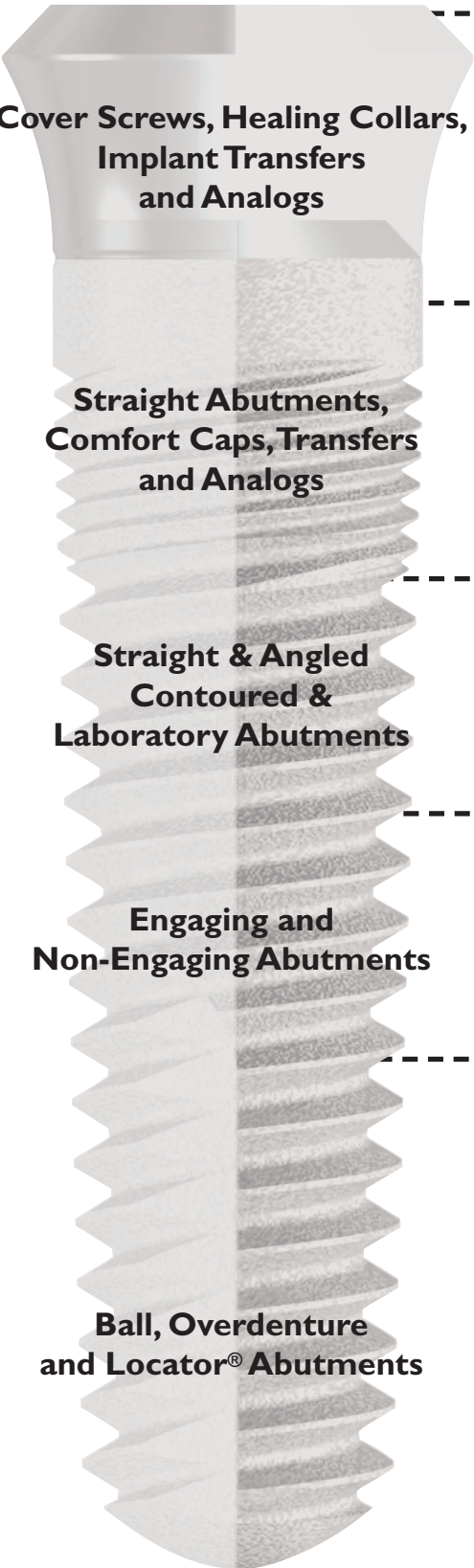

**Cover Screws, Healing Collars, Implant Transfers and Analogs**


**Straight Abutments, Comfort Caps, Transfers and Analogs**

**Straight & Angled Contoured & Laboratory Abutments**

**Engaging and Non-Engaging Abutments**

**Ball, Overdenture and Locator® Abutments**


**3.7mmD Platform**

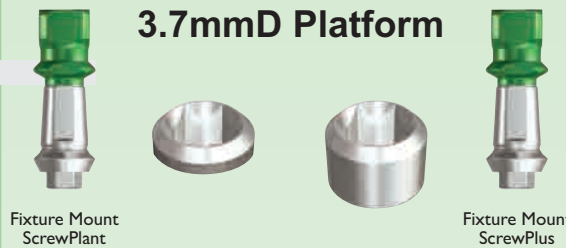

Fixture Mount ScrewPlant

Fixture Mount ScrewPlus

**4.7mmD**

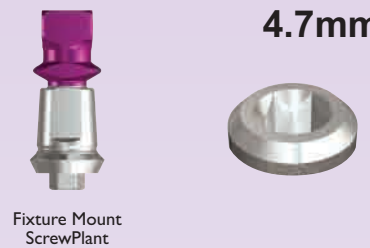

Fixture Mount ScrewPlant



1037-01


1037-13  
1037-13N

1037-15  
1037-15N

1037-05  
1037-05N

1037-08  
1037-08N

1037-06

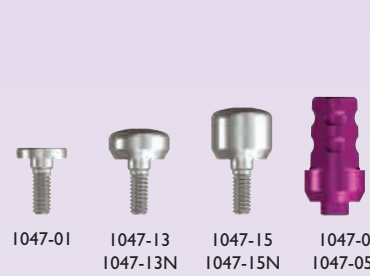

1047-01

1047-13  
1047-13N

1047-15  
1047-15N


1047-05  
1047-05N



1037-21N  
1037-22N

3037-26  
3037-02

3037

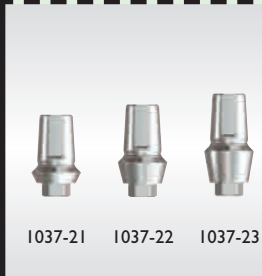

1037-21

1037-22

1037-23



1047-21N  
1047-22N

2047-20

3047-26  
3047-02

3047



1037-31

1037-32

1037-51

1037-52

1037-30L



1047-31

1047-32

1047-51



1037-40

1037-46


1037-47

1037-49



1047-40

1047-46

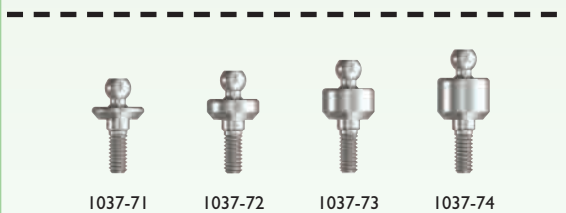

1037-71

1037-72

1037-73

1037-74



1047-71


1047-72



1037-61

1037-62

1037-64

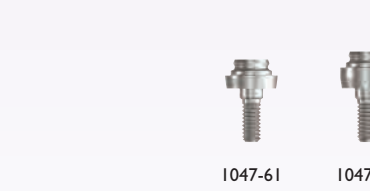

1047-61

1047-62



1037-91

1037-92

1037-93

1037-94



1047-91

1047-92

**Platform**



Fixture Mount  
ScrewPlus

**5.7mmD Platform**



Fixture Mount  
ScrewPlant

Fixture Mount  
ScrewPlus

**6.5mmD Platform**



Fixture Mount  
ScrewPlus



1047-08  
1047-08N

1047-06



1057-01

1057-13

1057-15

1057-05

1057-08

1057-06



2065-01

2065-13

2065-15

2065-05

2065-08

2065-06



1047-21

1047-22

1047-23



2057-20


3057-26  
3057-02

3057

1057-21

1057-22

1057-23

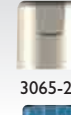

2065-20

3065-26  
3065-02

3065

3065



1047-52

1047-30L



1057-31

1057-32

1057-51

1057-52

1057-30L



1047-47

1047-49



1057-40

1057-46

1057-47

1057-49



2065-40

2065-46

2065-47

2065-49



1047-73

1047-74



1057-71

1057-72

1057-73

1057-74



1047-64



1057-61

1057-62

1057-64



1047-93



1047-94



1057-92

1057-93

1057-94



# Cover Screws, Healing Collars, Transfers & Analogs

## Replacement Surgical Cover Screws

Requires 1.25mmD Hex Tool

| Code    | Platform | Price   |
|---------|----------|---------|
| 1037-01 | 3.7mmD   | \$20.00 |
| 1047-01 | 4.7mmD   | \$20.00 |
| 1057-01 | 5.7mmD   | \$20.00 |
| 2065-01 | 6.5mmD   | \$20.00 |



## Healing Collars

Requires 1.25mmD Hex Tool

| Code     | Platform | Width  | Length | Price   |
|----------|----------|--------|--------|---------|
| 1037-13N | 3.7mmD   | 3.8mmD | 3mmL   | \$30.00 |
| 1037-13  | 3.7mmD   | 4.8mmD | 3mmL   | \$30.00 |
| 1037-15N | 3.7mmD   | 3.8mmD | 5mmL   | \$30.00 |
| 1037-15  | 3.7mmD   | 4.8mmD | 5mmL   | \$30.00 |
| 1047-13N | 4.7mmD   | 4.8mmD | 3mmL   | \$30.00 |
| 1047-13  | 4.7mmD   | 5.8mmD | 3mmL   | \$30.00 |
| 1047-15N | 4.7mmD   | 4.8mmD | 5mmL   | \$30.00 |
| 1047-15  | 4.7mmD   | 5.8mmD | 5mmL   | \$30.00 |
| 1057-13  | 5.7mmD   | 6.6mmD | 3mmL   | \$30.00 |
| 1057-15  | 5.7mmD   | 6.6mmD | 5mmL   | \$30.00 |
| 2065-13  | 6.5mmD   | 7.4mmD | 3mmL   | \$30.00 |
| 2065-15  | 6.5mmD   | 7.4mmD | 5mmL   | \$30.00 |



## Closed-Tray Implant Transfers

Includes Fixation Screw [1000-06]

| Code     | Platform | Replacement Screw | Price   |
|----------|----------|-------------------|---------|
| 1037-05N | 3.7mmD   | 1000-06           | \$30.00 |
| 1037-05  | 3.7mmD   | 1000-06           | \$30.00 |
| 1047-05N | 4.7mmD   | 1000-06           | \$30.00 |
| 1047-05  | 4.7mmD   | 1000-06           | \$30.00 |
| 1057-05  | 5.7mmD   | 1000-06           | \$30.00 |
| 2065-05  | 6.5mmD   | 1000-06           | \$30.00 |



## Open-Tray Implant Transfers

Includes Fixation Screw [1000-10]

| Code     | Platform | Replacement Screw | Price   |
|----------|----------|-------------------|---------|
| 1037-08N | 3.7mmD   | 1000-10           | \$30.00 |
| 1037-08  | 3.7mmD   | 1000-10           | \$30.00 |
| 1047-08N | 4.7mmD   | 1000-10           | \$30.00 |
| 1047-08  | 4.7mmD   | 1000-10           | \$30.00 |
| 1057-08  | 5.7mmD   | 1000-10           | \$30.00 |
| 2065-08  | 6.5mmD   | 1000-10           | \$30.00 |



## Implant Analogs

| Code    | Platform | Price   |
|---------|----------|---------|
| 1037-06 | 3.7mmD   | \$20.00 |
| 1047-06 | 4.7mmD   | \$20.00 |
| 1057-06 | 5.7mmD   | \$20.00 |
| 2065-06 | 6.5mmD   | \$20.00 |






# Straight Abutments, Snap-on Transfers and Lab Analogs

## Straight Abutments

Includes Transfer, Comfort Cap & Fixation Screw [1000-01]

| Code     | Platform | Collar Heights | Use Analog | Price   |
|----------|----------|----------------|------------|---------|
| 1037-21N | 3.7mmD   | 1mm            | 3037       | \$85.00 |
| 1037-22N | 3.7mmD   | 2mm            | 3037       | \$85.00 |
| 1037-21  | 3.7mmD   | 1mm            | 3047       | \$85.00 |
| 1037-22  | 3.7mmD   | 2mm            | 3047       | \$85.00 |
| 1037-23  | 3.7mmD   | 3mm            | 3047       | \$85.00 |
| 2047-20  | 4.7mmD   | 0mm            | 3047       | \$85.00 |
| 1047-21N | 4.7mmD   | 1mm            | 3047       | \$85.00 |
| 1047-22N | 4.7mmD   | 2mm            | 3047       | \$85.00 |
| 1047-21  | 4.7mmD   | 1mm            | 3057       | \$85.00 |
| 1047-22  | 4.7mmD   | 2mm            | 3057       | \$85.00 |
| 1047-23  | 4.7mmD   | 3mm            | 3057       | \$85.00 |
| 2057-20  | 5.7mmD   | 0mm            | 3057       | \$85.00 |
| 1057-21  | 5.7mmD   | 1mm            | 3065       | \$85.00 |
| 1057-22  | 5.7mmD   | 2mm            | 3065       | \$85.00 |
| 1057-23  | 5.7mmD   | 3mm            | 3065       | \$85.00 |
| 2065-20  | 6.5mmD   | 0mm            | 3065       | \$85.00 |

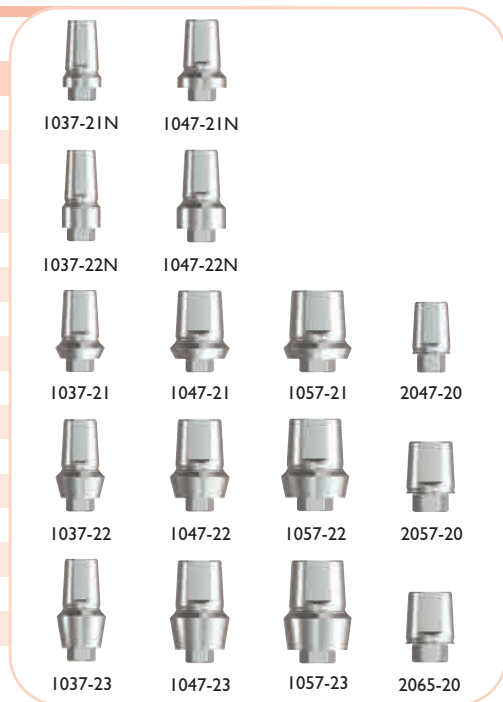

## Straight Abutment Transfers

| Code    | Abutment Width | Price (includes two pieces) |
|---------|----------------|-----------------------------|
| 3037-02 | 3.7mmD         | \$30.00                     |
| 3047-02 | 4.7mmD         | \$30.00                     |
| 3057-02 | 5.7mmD         | \$30.00                     |
| 3065-02 | 6.5mmD         | \$30.00                     |



## Straight Abutment Comfort Caps

| Code    | Platform | Price (includes two pieces) |
|---------|----------|-----------------------------|
| 3037-26 | 3.7mmD   | \$30.00                     |
| 3047-26 | 4.7mmD   | \$30.00                     |
| 3057-26 | 5.7mmD   | \$30.00                     |
| 3065-26 | 6.5mmD   | \$30.00                     |



## Straight Abutment Analogs

Includes Castable Coping

| Code | Platform | Price   |
|------|----------|---------|
| 3037 | 3.7mmD   | \$35.00 |
| 3047 | 4.7mmD   | \$35.00 |
| 3057 | 5.7mmD   | \$35.00 |
| 3065 | 6.5mmD   | \$35.00 |



## Straight Abutment Castable Copings

| Code    | Platform | Price (includes two pieces) |
|---------|----------|-----------------------------|
| 3037-28 | 3.7mmD   | \$30.00                     |
| 3047-28 | 4.7mmD   | \$30.00                     |
| 3057-28 | 5.7mmD   | \$30.00                     |
| 3065-28 | 6.5mmD   | \$30.00                     |






## Gold Engaging Cast-to Abutments

Includes Fixation Screw [1000-01]

| Code    | Platform | Collar Heights | Price    |
|---------|----------|----------------|----------|
| 1037-40 | 3.7mmD   | 1mm            | \$100.00 |
| 1047-40 | 4.7mmD   | 1mm            | \$100.00 |
| 1057-40 | 5.7mmD   | 1mm            | \$100.00 |
| 2065-40 | 6.5mmD   | 1mm            | \$100.00 |

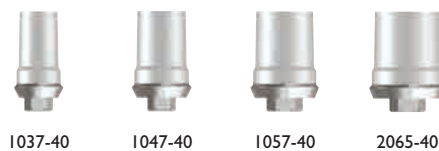

## Plastic Non-Engaging Castable Abutments

Includes Fixation Screw [1000-01]

| Code    | Platform              | Collar Heights | Price   |
|---------|-----------------------|----------------|---------|
| 1037-46 | 3.7mmD                | 0mm            | \$30.00 |
| 1047-46 | 4.7mmD                | 0mm            | \$30.00 |
| 1057-46 | 5.7mmD                | 0mm            | \$30.00 |
| 2065-46 | 6.5mmD                | 0mm            | \$30.00 |
| 4500-46 | GoDirect Implant only | 0mm            | \$30.00 |



## Plastic Engaging Temporary Abutments

Includes Fixation Screw [1000-06]


| Code    | Platform | Collar Heights | Price   |
|---------|----------|----------------|---------|
| 1037-47 | 3.7mmD   | 0mm            | \$35.00 |
| 1047-47 | 4.7mmD   | 0mm            | \$35.00 |
| 1057-47 | 5.7mmD   | 0mm            | \$35.00 |
| 2065-47 | 6.5mmD   | 0mm            | \$35.00 |



## Titanium Non-Engaging Temporary Abutments

Includes Fixation Screw [1000-01]

| Code    | Platform | Collar Heights | Price   |
|---------|----------|----------------|---------|
| 1037-49 | 3.7mmD   | 0mm            | \$35.00 |
| 1047-49 | 4.7mmD   | 0mm            | \$35.00 |
| 1057-49 | 5.7mmD   | 0mm            | \$35.00 |
| 2065-49 | 6.5mmD   | 0mm            | \$35.00 |

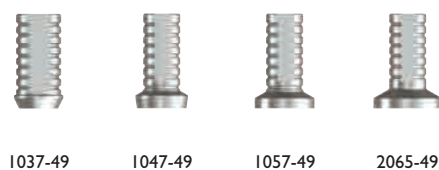





## Straight Full-Contoured Abutments

Includes Fixation Screw [1000-01]

| Code     | Platform | Collar Heights | Price   |
|----------|----------|----------------|---------|
| 1037-30L | 3.7mmD   | 1mm            | \$75.00 |
| 1047-30L | 4.7mmD   | 1mm            | \$75.00 |
| 1057-30L | 5.7mmD   | 1mm            | \$75.00 |



1037-30L



1047-30L



1057-30L

## Straight Contoured Abutments

Includes Fixation Screw [1000-01]

| Code    | Platform | Collar Heights | Price   |
|---------|----------|----------------|---------|
| 1037-31 | 3.7mmD   | 1mm            | \$85.00 |
| 1037-32 | 3.7mmD   | 2mm            | \$85.00 |
| 1047-31 | 4.7mmD   | 1mm            | \$85.00 |
| 1047-32 | 4.7mmD   | 2mm            | \$85.00 |
| 1057-31 | 5.7mmD   | 1mm            | \$85.00 |
| 1057-32 | 5.7mmD   | 2mm            | \$85.00 |



1037-31



1047-31



1057-31



1037-32



1047-32



1057-32

## Angled Contoured Abutments

Includes Fixation Screw [1000-01]

| Code    | Platform | Collar Heights | Price   |
|---------|----------|----------------|---------|
| 1037-51 | 3.7mmD   | 1mm            | \$85.00 |
| 1037-52 | 3.7mmD   | 2mm            | \$85.00 |
| 1047-51 | 4.7mmD   | 1mm            | \$85.00 |
| 1047-52 | 4.7mmD   | 2mm            | \$85.00 |
| 1057-51 | 5.7mmD   | 1mm            | \$85.00 |
| 1057-52 | 5.7mmD   | 2mm            | \$85.00 |



1037-51



1047-51



1057-51



1037-52



1047-52



1057-52

## Replacement Fixation Screws

| Code    | Description          | Price (includes two pieces) |
|---------|----------------------|-----------------------------|
| 1000-01 | Short Fixation Screw | \$30.00                     |
| 1000-06 | Fixation Screw       | \$30.00                     |
| 1000-10 | Long Fixation Screw  | \$30.00                     |

Abutments



1000-01

Closed-Tray Transfer & Temporary Abutment



1000-06










































Open-Tray Transfer & Waxing Screw



1000-10



# Selection Chart: Overdenture & Screw-Retained Restorations

|  | 3.7mmD Platform   | 4.7mmD Platform   | 5.7mmD Platform  | 6.5mmD Platform  |
|--|---|---|--|--|
|  |   |   |  |  |
|  | <b>Overdenture Abutments</b> (*Includes Transfer and Comfort Cap)   |   |  |  |
| Overdenture Abutments                        | <br>1037-61<br>1037-62<br>1037-64              | <br>1047-61<br>1047-62<br>1047-64              | <br>1057-61<br>1057-62<br>1057-64              |  |
|  | <b>Overdenture Abutment Transfers, Analog &amp; Comfort Cap</b>   |   |  |  |
| OD Abutment Comfort Cap Transfers and Analog | <br>*4000-13                                   | <br>*1000-68                                   | <br>1000-61                                     | <br>1000-62                 |
|  |   |   | <br>Open-Tray Transfer                        | <br>1000-63                 |
|  |   |   |  | <br>Closed-Tray Transfer    |
|  | <b>Overdenture Copings</b>  |   |  |  |
| Overdenture Abutment Components              | <br>1000-64<br>Gold                            | <br>1000-65<br>Titanium                        | <br>1000-66<br>Plastic                          | <br>1000-67<br>Gold/Plastic |
|  |   |   |  | <br>1000-69<br>Titanium     |
|  | <b>Ball Abutments</b> (**Includes Transfer and Comfort Cap)   |   |  |  |
| Ball Abutments                               | <br>1037-71<br>1037-72<br>1037-73<br>1037-74 | <br>1047-71<br>1047-72<br>1047-73<br>1047-74 | <br>1057-71<br>1057-72<br>1057-73<br>1057-74 |  |
|  | <b>Ball Attachments</b>   |   |  |  |
| Ball Abutment Components                     | <br>**1000-87                                | <br>**1000-84                                | <br>1000-85                                   | <br>1000-81                 |
|  |   |   | <br>1000-82                                   | <br>1000-88               |
|  |   |   |  | <br>1000-83               |
|  |   |   |  | <br>1000-86               |
|  | <b>Locator® Abutments</b>   |   |  |  |
| Locator® Abutments                           | <br>1037-91<br>1037-92<br>1037-93<br>1037-94 | <br>1047-91<br>1047-92<br>1047-93<br>1047-94 | <br>1057-92<br>1057-93<br>1057-94            |  |
|  | <b>Locator® Attachments</b>   |   |  |  |
| Locator® Abutment Components                 | <br>1000-94                                  | <br>1000-95                                  | <br>1000-95W                                  | <br>1000-92B                |
|  |   |   | <br>1000-92C                                  | <br>1000-92G                |
|  |   |   |  | <br>1000-92P                |
|  |   |   |  | <br>1000-92R                |
|  |   |   |  | <br>1000-98               |
|  |   |   |  | <br>1000-93               |
|  |   |   |  | <br>1000-96               |




# Abutments for Ball, Bar & Locator® Overdentures

## Overdenture Abutments

Includes Transfer & Comfort Cap  
[Requires 1.25mmD Hex Tool]


| Code    | Platform | Collar Heights | Price   |
|---------|----------|----------------|---------|
| 1037-61 | 3.7mmD   | 1mm            | \$85.00 |
| 1037-62 | 3.7mmD   | 2mm            | \$85.00 |
| 1037-64 | 3.7mmD   | 4mm            | \$85.00 |
| 1047-61 | 4.7mmD   | 1mm            | \$85.00 |
| 1047-62 | 4.7mmD   | 2mm            | \$85.00 |
| 1047-64 | 4.7mmD   | 4mm            | \$85.00 |
| 1057-61 | 5.7mmD   | 1mm            | \$85.00 |
| 1057-62 | 5.7mmD   | 2mm            | \$85.00 |
| 1057-64 | 5.7mmD   | 4mm            | \$85.00 |

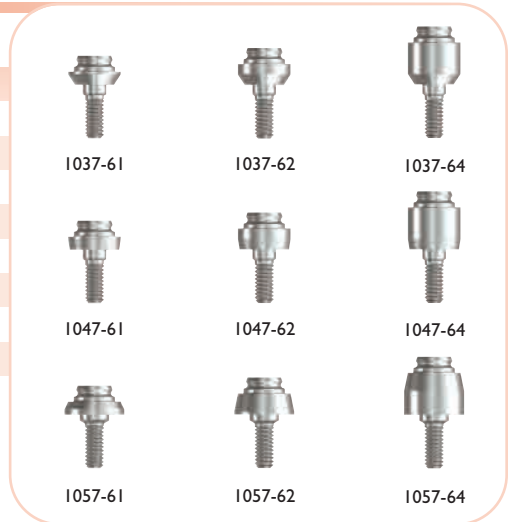

## Ball Abutments

Includes Transfer & Comfort Cap  
[Requires 1.25mmD Hex Tool]


| Code    | Platform | Collar Heights | Price   |
|---------|----------|----------------|---------|
| 1037-71 | 3.7mmD   | 1mm            | \$85.00 |
| 1037-72 | 3.7mmD   | 2mm            | \$85.00 |
| 1037-73 | 3.7mmD   | 3mm            | \$85.00 |
| 1037-74 | 3.7mmD   | 4mm            | \$85.00 |
| 1047-71 | 4.7mmD   | 1mm            | \$85.00 |
| 1047-72 | 4.7mmD   | 2mm            | \$85.00 |
| 1047-73 | 4.7mmD   | 3mm            | \$85.00 |
| 1047-74 | 4.7mmD   | 4mm            | \$85.00 |
| 1057-71 | 5.7mmD   | 1mm            | \$85.00 |
| 1057-72 | 5.7mmD   | 2mm            | \$85.00 |
| 1057-73 | 5.7mmD   | 3mm            | \$85.00 |
| 1057-74 | 5.7mmD   | 4mm            | \$85.00 |

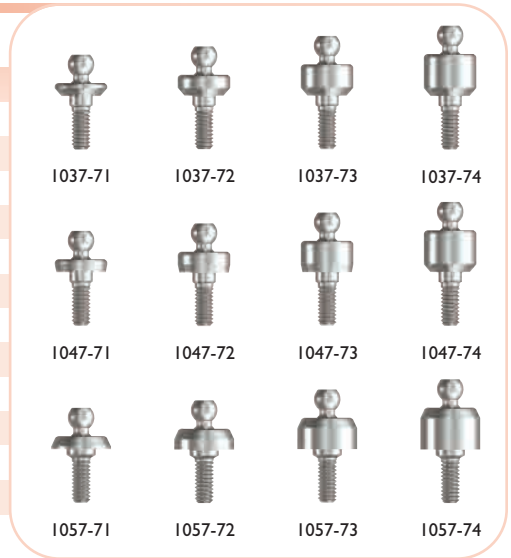

## Overdenture and Ball Abutment for GoDirect Implant

Includes Transfer & Comfort Cap  
[Requires 1.25mmD Hex Tool]


| Code    | Platform | Collar Heights | Price   |
|---------|----------|----------------|---------|
| 4500-61 | 3.7mmD   | 1mm            | \$85.00 |
| 4500-71 | 3.7mmD   | 1mm            | \$85.00 |

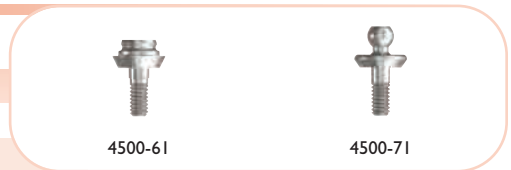

## Locator® Abutments

Requires Abutment Processing Cap/Male Kit [1000-98]

| Code    | Platform | Collar Heights | Price    |
|---------|----------|----------------|----------|
| 1037-91 | 3.7mmD   | 1mm            | \$110.00 |
| 1037-92 | 3.7mmD   | 2mm            | \$110.00 |
| 1037-93 | 3.7mmD   | 3mm            | \$110.00 |
| 1037-94 | 3.7mmD   | 4mm            | \$110.00 |
| 1047-91 | 4.7mmD   | 1mm            | \$110.00 |
| 1047-92 | 4.7mmD   | 2mm            | \$110.00 |
| 1047-93 | 4.7mmD   | 3mm            | \$110.00 |
| 1047-94 | 4.7mmD   | 4mm            | \$110.00 |
| 1057-92 | 5.7mmD   | 2mm            | \$110.00 |
| 1057-93 | 5.7mmD   | 3mm            | \$110.00 |
| 1057-94 | 5.7mmD   | 4mm            | \$110.00 |

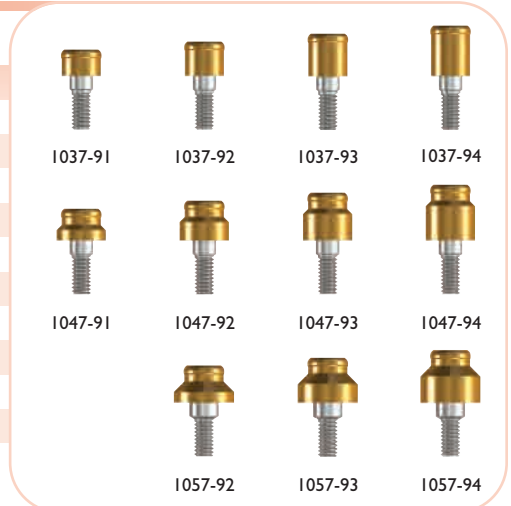




# Ancillary Components for Overdenture & Screw-Retained Restorations

## Overdenture Abutment Comfort Cap, Transfers & Analog Screws require 1.25mmD Hex Tool

| Code    | Platform | Description                                  | Price   |
|---------|----------|--|---------|
| 1000-61 | 5.0mmD   | Aluminum Analog                              | \$20.00 |
| 1000-62 | 5.0mmD   | Open-Tray Transfer plus Fixation Screw       | \$30.00 |
| 1000-63 | 5.0mmD   | Closed-Tray Transfer plus Fixation Screw     | \$30.00 |
| 1000-68 | 5.0mmD   | Plastic Abutment Transfer - 2ea              | \$30.00 |
| 4000-13 | 5.0mmD   | Plastic Comfort Cap - 2ea                    | \$30.00 |
| 1000-14 | -        | Replacement Open-Tray Fixation Screw - 2ea   | \$30.00 |
| 1000-15 | -        | Replacement Closed-Tray Fixation Screw - 2ea | \$30.00 |

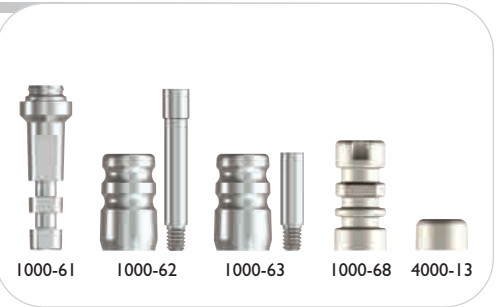


## Overdenture Abutment Components Includes Fixation Screw [1000-05]

| Code    | Platform | Description                 | Price   |
|---------|----------|-----------------------------|---------|
| 1000-64 | 5.0mmD   | Gold Coping                 | \$80.00 |
| 1000-65 | 5.0mmD   | Titanium Coping             | \$40.00 |
| 1000-66 | 5.0mmD   | Plastic Coping              | \$35.00 |
| 1000-67 | 5.0mmD   | Gold & Plastic Coping       | \$80.00 |
| 1000-69 | 5.0mmD   | Titanium Temporary Coping   | \$35.00 |
| 1000-05 | -        | Coping Fixation Screw - 2ea | \$30.00 |



## Ball Abutment Components

| Code    | Description                         | Price   |
|---------|-------------------------------------|---------|
| 1000-81 | Ball Abutment Housing - 2ea         | \$30.00 |
| 1000-82 | Nylon Liner - 2ea                   | \$15.00 |
| 1000-83 | Nylon Liner Transfer - 2ea          | \$15.00 |
| 1000-84 | Ball Abutment Transfer - 2ea        | \$30.00 |
| 1000-85 | Ball Abutment Analog                | \$20.00 |
| 1000-86 | Ball Abutment Tools                 | \$65.00 |
| 1000-87 | Ball Abutment Comfort Cap - 2ea     | \$30.00 |
| 1000-88 | Ball Abutment Housing & Nylon Liner | \$22.00 |

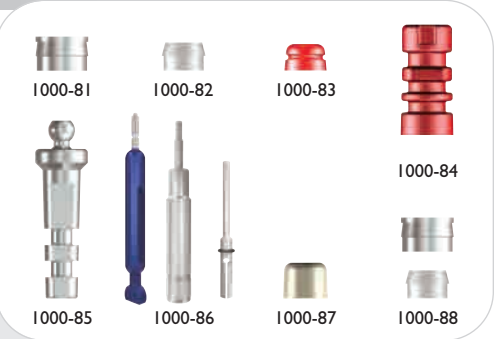

## Locator® Abutment Transfer, Analogs and Alignment Tools

| Code     | Description  | Price   |
|----------|--|---------|
| 1000-94  | Locator® Impression Coping - 2ea                             | \$32.00 |
| 1000-95  | Locator® Abutment Analog 4mmD (for 3.5 & 4.3mmD) - 2ea       | \$32.00 |
| 1000-95W | Locator® Abutment Analog 5mmD (for 5.0mmD) - 2ea             | \$32.00 |
| 1000-97  | Locator® Abutment Parallel Post - 4ea                        | \$16.00 |
| 1000-10  | Guide Screw for ScrewPlant/ScrewPlus implant platforms - 2ea | \$30.00 |
| 1000-99  | Locator® Abutment Angle Measure Guide                        | \$21.00 |



## Locator® Abutment Components

| Code     | Retention Force | Description                               | Price   |
|----------|-----------------|---|---------|
| 1000-92C | 5.0lbs          | Replacement Male Clear - 4ea              | \$21.00 |
| 1000-92P | 3.0lbs          | Light Replacement Male Pink - 4ea         | \$21.00 |
| 1000-92B | 1.5lbs          | Extra Light Replacement Male Blue - 4ea   | \$21.00 |
| 1000-92G | 4.0lbs          | Extended Range Male Green - 4ea           | \$21.00 |
| 1000-92R | 1.0lb           | Extra Light Extended Range Male Red - 4ea | \$21.00 |
| 1000-93  | -               | Black Processing Male - 4ea               | \$21.00 |
| 1000-98  | Assorted        | Abutment Processing Cap/Male Kit - 2ea    | \$58.00 |



## Locator® Instrumentation

| Code    | Description   | Price   |
|---------|---|---------|
| LDT     | Locator® Abutment Driver Tool - Long                        | \$44.00 |
| LDTs    | Locator® Abutment Driver Tool - Short                       | \$44.00 |
| LDD     | Locator® Abutment Driver Drill - Long                       | \$44.00 |
| LDdS    | Locator® Abutment Driver Drill - Short                      | \$44.00 |
| 1000-96 | Core Tool - Male Removal & Seating Tool and Abutment Driver | \$75.00 |





## Tapered Hex Tools

| Code     | Description                    | Price   |
|----------|--------------------------------|---------|
| THT1.25  | 1.25mmD Tapered Hex Tool Long  | \$40.00 |
| THT1.25S | 1.25mmD Tapered Hex Tool Short | \$40.00 |



THT1.25



THT1.25S

## Straight Hex Tools

*Square connection for use with Ratchet or Torque Wrench*

| Code    | Description                                   | Price   |
|---------|---|---------|
| HT1.25  | 1.25mmD Straight Hex Tool Long                | \$40.00 |
| HT1.25S | 1.25mmD Straight Hex Tool Short               | \$40.00 |
| HT2.5   | 2.5mmD Hex Tool Long (with retentive O-ring)  | \$40.00 |
| HT2.5S  | 2.5mmD Hex Tool Short (with retentive O-ring) | \$40.00 |



HT1.25



HT2.5



HT1.25S



HT2.5S

## Hex Drills

*Latch head connection for use with Surgical Motor*

| Code   | Description                              | Price   |
|--------|--|---------|
| HD1.25 | 1.25mmD Hex Drill                        | \$40.00 |
| HD2.5  | 2.5mmD Hex Drill (with retentive O-ring) | \$40.00 |



HD1.25



HD2.5

## Ancillary Instruments

| Code | Description  | Price    |
|------|--|----------|
| TW30 | 30Ncm Torque Wrench  | \$125.00 |
| TW35 | 35Ncm Torque Wrench  | \$125.00 |
| DE   | Drill Extender   | \$40.00  |
| TWM  | Multi-Setting Torque Wrench (15, 20, 25, 30, 35, 40, 45, 50 & 60Ncm) | \$395.00 |
| ABH  | Analog Holder  | \$85.00  |



TW30



TW35



DE



TWM



ABH



**Implant Direct™**

simply smarter.

Implant Direct International Inc.  
8840 Russell Road,  
Las Vegas, NV 89148  
Manufacturing/Sales: 1-818-444-3333  
www.implantdirect.com



**“Order Online for High Quality Products at factory-direct prices”**

Gerald Niznick, DMD, MSD, Founder and President of Implant Direct

## ScrewPlant Implant

- All-in-1 Packaging for Simplicity and Economy
- 2mm Deep Internal Hex with External Bevel
- Double-lead Threads for Faster Insertion
- Micro-threads for Increased Stability
- Surface Options: SBM or HA Coating



## Spectra-System®

Application-Specific Implants include Abutment, Transfer, Cover Screw & Healing Collar/Comfort Cap  
Six Implants - One Surgical Protocol - \$150/€115

### Two-Piece Implants

2-Stage



ScrewPlant®  
Internal Hex

1-Stage



ScrewPlus®  
Internal Hex



**12 Abutment Options**  
Titanium: \$75 & \$85  
Gold: \$100  
Locator®: \$110

### One-Piece Implants



ScrewDirect®  
Straight Snap-on  
available in 3.0mmD



GoDirect™  
OD Attachment  
available in 3.0mmD



ScrewIndirect®  
Screw-Receiving  
available in 3.0mmD



ScrewRedirect®  
Angled Contoured

**Free Implants for Zimmer Dental Customers**  
3 ScrewPlant Implants with Fixture-mount/Abutments  
Compatible with Zimmer Dental's Tapered Screw-Vent® Drills

US List Price for 3 Implants, Abutments and Prosthetic Components:

**ScrewPlant**

= \$540

**Zimmer®**

= \$1868



**FREE Introductory Offers at**  
[www.implantdirect.com](http://www.implantdirect.com)

Terms and Conditions Apply