

Alpha-Bio TEC®

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Alpha-Bio TEC®	SPI/DFI/ATDI 3.75	20 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Astra Tech	Multi Unit 20°	15 Ncm	10 – 12 Ncm
Astra Tech	Multi Unit 45°	15 Ncm	10 – 12 Ncm
Astra Tech	OsseoSpeed 3.0	15 Ncm	10 – 12 Ncm
Astra Tech	OsseoSpeed 3.5/4.0	20 Ncm	10 – 12 Ncm
Astra Tech	OsseoSpeed 4.5/5.0	20 Ncm	10 – 12 Ncm

Biomet 3i™

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Biomet 3i™	Certain® 3.4	20 Ncm	10 – 12 Ncm
Biomet 3i™	Certain® 4.1	20 Ncm	10 – 12 Ncm
Biomet 3i™	Certain® 5.0	20 Ncm	10 – 12 Ncm
Biomet 3i™	Certain® 6.0	20 Ncm	10 – 12 Ncm
Biomet 3i™	OSSEOTITE® 3.4	35 Ncm	10 – 12 Ncm
Biomet 3i™	OSSEOTITE® 4.1	35 Ncm	10 – 12 Ncm
Biomet 3i™	OSSEOTITE® 5.0	35 Ncm	10 – 12 Ncm
Biomet 3i™	OSSEOTITE® 6.0	35 Ncm	10 – 12 Ncm

Bredent SKY

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Bredent SKY	fast & fixed	20 Ncm	11 – 12 Ncm
Bredent SKY	Classic/blueSKY	25 Ncm	12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
BTI®	Conical Spacer 4.1 – 5.5	10 Ncm	10 – 12 Ncm
BTI®	Externa® 4.1 – 5.5	20 Ncm	10 – 12 Ncm
BTI®	Interna®	20 Ncm	10 – 12 Ncm
BTI®	Multi-IM 3.5/4.1/5.5/Angled	10 Ncm	10 – 12 Ncm
BTI®	Tiny®	20 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Camlog®	J-Type/K-Type 3.3	20 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 3.8	20 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 4.3	30 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 5.0	30 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 6.0	30 Ncm	10 – 12 Ncm
Camlog®	Multi Unit Abutment 3.3/3.8/4.3	20 Ncm	10 – 12 Ncm
Camlog®	Multi Unit Abutment 5.0/6.0	20 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Dyna®	Octalock®/Helix®	35 Ncm	11 – 12 Ncm

Friadent DENTSPLY

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Friadent DENTSPLY	XiVE® 3.4	25 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® 3.8	25 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® 4.5	25 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® 5.5	25 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® MP/TG 3.4/3.8/4.5/5.5	25 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
ICX®	ICX®-templant®	25 Ncm	11 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
MIS®	Standard	30 Ncm	11 – 12 Ncm
MIS®	Wide	30 Ncm	11 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Nobel Biocare®	Brånemark® 3.5	25 Ncm	10 – 12 Ncm
Nobel Biocare®	Brånemark® 4.1	35 Ncm	10 – 12 Ncm
Nobel Biocare®	Brånemark® 5.1	45-50 Ncm	10 – 12 Ncm
Nobel Biocare®	Multi Unit Abutment NP-RP	15 Ncm	10 – 12 Ncm
Nobel Biocare®	Multi Unit Abutment WP	35 Ncm	10 – 12 Ncm
Nobel Biocare®	NobelReplace® 3.5	35 Ncm	10 – 12 Ncm
Nobel Biocare®	NobelReplace® 4.3	35-45 Ncm	10 – 12 Ncm
Nobel Biocare®	NobelReplace® 5.0	35-45 Ncm	10 – 12 Ncm
Nobel Biocare®	NobelReplace® 6.0	35-45 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Nobel Biocare®	Nobel Active® NP	25 Ncm	10 – 12 Ncm
Nobel Biocare®	Nobel Active® RP	35 – 45 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
SIC®	SiCace 3.3	20 Ncm	10 – 12 Ncm
SIC®	SiCace 4.2	20 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Straumann®	Bone Level® NC	20 Ncm	10 – 12 Ncm
Straumann®	Bone Level® RC	20 Ncm	10 – 12 Ncm
Straumann®	synOcta® 3.5	35 Ncm	10 – 12 Ncm
Straumann®	synOcta® 4.8	35 Ncm	10 – 12 Ncm
Straumann®	synOcta® 6.5	35 Ncm	10 – 12 Ncm

Sweden & Martina

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Sweden & Martina	Kohno/Premium 3.3	30 Ncm	10 – 12 Ncm
Sweden & Martina	Kohno/Premium 3.8	30 Ncm	10 – 12 Ncm
Sweden & Martina	Kohno/Premium 4.25	35 Ncm	10 – 12 Ncm
Sweden & Martina	Kohno/Premium 5.0	35 Ncm	10 – 12 Ncm
Sweden & Martina	Kohno/Premium 6.0	35 Ncm	10 – 12 Ncm
Sweden & Martina	Multi Unit Abutment	15 Ncm	10 – 12 Ncm
Sweden & Martina	Out-Link 3.3	30 Ncm	10 – 12 Ncm
Sweden & Martina	Out-Link 4.1	35 Ncm	10 – 12 Ncm
Sweden & Martina	Out-Link 5.0	35 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Zimmer®	Tapered Screw-Vent® 3.5	30 Ncm	10 – 12 Ncm
Zimmer®	Tapered Screw-Vent® 4.5	30 Ncm	10 – 12 Ncm
Zimmer®	Tapered Screw-Vent® 5.7	30 Ncm	10 – 12 Ncm