

# Molloplast® B

Permanently soft relining material for comfortable, long-term wear




- **Molloplast® B**, single component silicone comes ready to use, eliminating mixing and dosing errors.
- Easy to use with dental flasks: can be polymerized either in a water bath or using microwaves.
- Time-saving feature: can be polymerized simultaneously with acrylates.
- Bonds firmly to all types of denture acrylates, to newly made as well as pre-worn denture bases.
- Due to its specially formulated components, **Molloplast® B** stands up to the influences of the oral environment long-term.
- **Molloplast® B** is non-irritant, tissue compatible, clinically tested and scientifically documented.
- **Molloplast® B** is odour- and tasteless.
- Dentures relined with **Molloplast® B** stay comfortable for years of wearing and fully meet the demands of modern dental prosthetics.

clinically tested &  
scientifically documented

# Molloplast® B

Single-component silicone, homogeneous consistency – heat curing, permanently soft

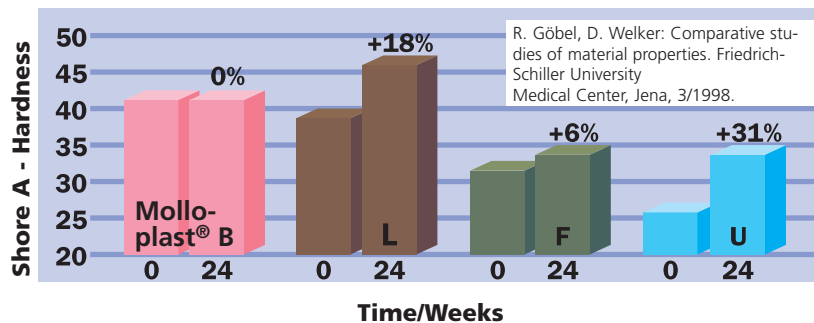

■ **Molloplast® B** contains no plasticizers and therefore will not change its consistency secondary to leaching.

■ Dentures relined with **Molloplast® B** stay soft and elastic for years.



■ **Molloplast® B** is suited for an especially large range of uses and can be used for relining, for covering of defect areas in the mouth, and for rebasing. **Molloplast® B** is also excellent for mouth guards, night-grinding guards and for modelling individual palate contours.

■ Dentures relined with **Molloplast® B** stay comfortable for years of wearing and fully meet the demands of modern dental prosthetics.



■ **Molloplast® B** has a dough-like, homogeneous consistency and comes ready to use. It can be polymerized simultaneously with acrylate when making new dentures, but can also be used for relining of already-worn dentures.

■ Once polymerized, **Molloplast® B** provides a highly elastic prosthetic base with a rubbery consistency that bonds absolutely firmly to the acrylate.



■ **Molloplast® B** is wetted completely saliva. Due to its high elasticity, it moulds intimately to the gingival contours during chewing; it thus constantly maintains the capillary film of saliva at the denture base that is so crucial for proper denture adhesion.

■ **Molloplast® B** contains no substances capable of swelling and therefore will not change shape by swelling.

■ Due to its specially formulated components, **Molloplast® B** stands up to the influences of the oral environment and resists bacteria and saliva.

■ **Molloplast® Remover** for fast and secure dissolution of the bonding between elastic relining materials and the denture acrylic. Simple application, without any time-consuming removing by grinding. No residues, no interaction during repeated relinings.



## Indications for use:

Elastic cover of sharp-ridged alveolar processes with a thin tegument.

Cushioning of dentures against the mylohyoid line, mental foramen or other prominences of the alveolar ridge such as the retromolar triangle or tubera.

Lower jaws with a "tram-track"-profile.

Cushioning against the median suture.

Post-damming of the A line and to increase upper and lower denture stability by improved adhesion.

Construction of bracketless dentures and for covering congenital or postoperative defect areas with solid or inflatable obturators.

## Processing technique:

**Molloplast® B** is simple to use. It comes ready to use and has a dough-like consistency. It can be extruded directly onto hard acrylic denture parts that have been pre-pressed with a spacer.

Once polymerized according to the instructions, **Molloplast® B** is firmly bonded to the denture acrylic. Pre-worn dentures can also be relined at any time with **Molloplast® B**.

## Ordering information:

Standard packing 03000  
45 g Molloplast® B

Laboratory packing 03001  
170 g Molloplast® B

Combination packing 03002  
45 g Molloplast® B  
15 ml Primo adhesive liquid

Primo adhesive liquid 03004  
15 ml bottle

LUSTROL gloss varnish  
Combination packing 03008  
6 ml LUSTROL gloss varnish  
6 ml catalyst

Molloplast®-grinding tools  
1 set (3 pcs.) 03101  
1 set (2 pcs.) 03120

Molloplast®-pre-polishers 03153  
1 set (10 pcs. + 1 mandrell)

10 grinding sleeves 02615

1 grinding sleeve holder 02616

# DETAX

GmbH & Co. KG  
Carl-Zeiss-Straße 4 · 76275 Ettlingen  
Telefon 07243 / 510-0 · Fax 07243 / 510-100  
www.detax.de · post@detax.de

Made in Germany

