Deutsch

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Compiled: 02.02.2012

Working instructions:

- 1) Mill resin discs while taking into account specific parameters for resin milling
- 2) Check milled framework for obvious chips or cracks, use only fault-free framework for temporary restorations.
- 3) Mechanical trimming of blanks is possible
- 4) Free object from dust
- 5) Use temporary, eugenol-free cements for insertion

Hints:

Resin discs are already polymerized when supplied, but avoid dropping them nevertheless. Product quality and durability can be compromised if handled improperly. Dust from grinding can irritate eyes, skin or mucous membranes, take protective measures.

Should intolerance to the product be suspected, use it only after an allergy screening test has been performed. Disinfect the milled parts before inserting them in the patient's mouth. Implant hygiene is the patient's responsibility. These instructions for use are based on our current levels of knowledge.

The patient will be held fully responsible as regards correct use of the product. The manufacturer is not liable for any unsatisfactory results, given that it has no influence on processing procedures. The dental technician shall inform the dentist as regards the operations to be performed (such as sterilization for example) before inserting the part into the patient's mouth, making sure that the dentist has received all of the information contained in these instructions for use. Any claims for damages shall concern exclusively the commercial value of our products.

Dimension:

- Wall thickness (cervical) min. 0,7 mm
- Wall thickness (occlusal) min. 1,2 mm
- Cross section connecting bar anterior tooth: min 12 mm²
- Cross section connecting bar posterior teeth min 16 mm²
- Pontics max. 2

Resistance:

	E-Module	Bending strenght
Basic	1800 MPa	70 MPa
Premium	3400 MPa	106 MPa
Premium Flexible	2400 MPa	100 MPa
Premium Antiba	2400 MPa	100 MPa

Storage and durability:

Store in a dry place at room temperature, do not expose to physical impact or vibration, avoid contact with liquids, protect discs from contamination.

Handling:

Handling of this product only by trained personnel. Avoid any exposure to humidity. Avoid exposure to dust. Ensure adequate ventilation.

Use in accordance with regulations:

TEMP is indicated for the manufacture of provisional anterior / posterior crown & bridge (TEMP BASIC - max. 6 months, all types of TEMP PREMIUM - max. 1 year) framework.

Contraindications:

TEMP is incompatible with some herbal and medicinal types of tea containing essential oils, as well as with some spices. Thus, cloves, nutmeg and pimento can damage TEMP. Fennel tea may contain essential oils in sufficient amounts to let TEMP swell, as it is being prepared from decoction of fruit.

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