Deutsch

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Gebrauch:

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Date of update: 29.02.2012

Indication:

- 1) Pre-mill 'green' zirconia (diameter 2.31 – depth 4.12) for threaded bush with
- 2) Cut thread into zirconia with a suitable tapping tool
- 3) Sinter frame work
- 4) Cement bush with commercially available cement. Ensure housing does not protrude zirconia frame
- 5) Design secondary parts using aides provided (i.e. Extension tubes, software programs, etc)
- 6) Construct secondary parts (zirconia milling and sintering)
- 7) Join primary and secondary parts using suitable screw
- 8) Fasten screw with 20 N·cm

Hints:

Ensure even flat contacts when screw-joining parts: No 'high spot' uneven contacts!
Primary and secondary parts must flush-fit together. Primary and secondary parts must be designed to allow possible lateral forces on the zirconia frame alone but not the actual screw connection itself.
Only use Zirkonzahn products!
This device is used by dental technicians in the dental laboratory. Only to be used by specialized technicians for the realization of complex dental prostheses to replace anatomical parts of the human body which are worn, severely damaged or which need to be replaced. The device is must not get in touch with tissue, bone or dentin.
Find detailed instructions and picture manuals under www.zirkonzahn.com.

Eye Protection:

Wear protective glasses

Respiratory Protection:

Wear face mask suitable for filtering fine air-borne dust particles. Avoid uninterrupted continuous use of system's air brush. Use in short bursts only to prevent unnecessary dust pollution! Ensure adequate ventilation.

Storage and Conservation:

Store in dry place

Further Information:

Manufacturer: Zirkonzahn S.r.l.
An der Ahr 7 - 39030 Gais - Italy
www.zirkonzahn.com
Supplier: Zirkonzahn S.r.l.
An der Ahr 7 - 39030 Gais - Italy
www.zirkonzahn.com
Emergency phone: +39 0474 066 660
(Mon. - Fri. 8 am - 6 pm)

ZZ Screw-Tec was designed for use in conjunction with ICE ZIRKON zirconia or PRETTAU ZIRKON zirconia.

N.B.: All medical products are non-returnable.

Gebrauchsanweisung - Informazione d'uso
Instruction manual - Istruções de uso
Mode d'emploi

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