


*Fits like a glove: the NEW premium temporary*

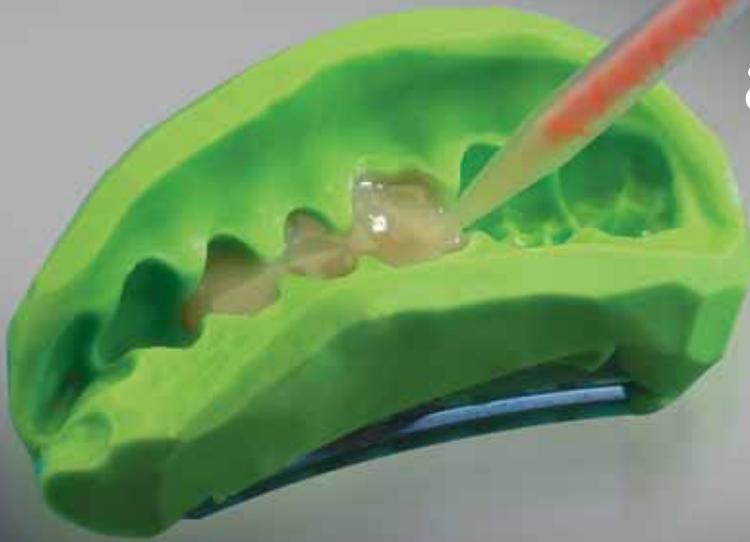

almost too good  
to be true



## tempofit® premium

### self curing 10:1 composite for premium provisionals

Top formulation with exemplary material properties due to a novel resin matrix, for high aesthetic short and long term provisionals

- **exceptional bending strength**, highest pressure and abrasion resistance
- **exactly fitted** and unstressed due to a slightest polymerisation shrinkage
- **minimal polymerisation temperature** during setting
- **defined elastic phase** for reliable removal of the provisional
- brilliant, translucent colours for a exceptional **natural aesthetic** and fluorescence, even in different lighting conditions
- **absolutely colour stable** for complete time of wearing, tasteless
- easy surface finishing, excellent grinding and polishing

new



# tempofit® premium

## temporary crown and bridge material 10:1, self curing

- colour-stable and polishable to a high sheen, providing for a natural aesthetic. Its colouration can be individually adjusted, compatible with UV-light-polymerising composites
- low polymerisation temperatures prevent pulpal and gingival irritation
- for direct application in Automix2 cartridge system, ideal mixing ratio 10:1

### smartrepair® – light curing composite



in handy application syringes, with exchangeable tips that allow exact application. Even complicated repairs, completing crown edges and anatomical contouring are easy and quick to perform. Ideal for intra- and extra-oral use.

smartbond® – one-step bonding – is applied to the area to be repaired in one single step and can be polymerised simultaneously with smartrepair® under UV-light.

## tempolink®

### temporary cements for crowns and bridges

#### tempolink® single/multi

- polycarboxylate-based, hydrophilic cement, self curing, eugenol-free
- secure sealing without any marginal leakage, maximum adhesion
- no dehydration of the preparation, no penetration of bacteria
- antibacterial, of indifferent colour and odour
- **single – 60 sec.** for the fabrication of single crowns and smaller applications
- **multi – 150 sec.** for the fabrication of multipart provisionals, resp. larger restorations

also available transparent:

#### tempolink® clear

- composite-based
- dual curing system: self curing & light curing
- flexible working times
- especially suitable for anterior tooth region
- low displacement resistance, easy cementing



## DETAX

GmbH & Co. KG  
Carl-Zeiss-Straße 4 · 76275 Ettlingen  
Telefon 07243 / 510-0 · Fax 07243 / 510-100  
www.detax.de · post@detax.de

Made in Germany



#### Indications for use:

Fabrication of crown and bridges temporaries for short- or long-term use. Protection of preparation area from influences of oral environment.

#### Processing technique:

Before beginning preparative work, a situation impression can be made using silicone or thermo-plastic material. Apply tempofit® premium directly from the cartridge into the situation impression. The material is easy to remove during the elastic phase of polymerisation and can be trimmed and polished to a high sheen with minimal delay. Repairs and corrections can be done easily and at minimal time cost by directly applying the UV-light curing smartrepair® system repair material. For temporary attachment of crown and bridge temporaries we recommend using tempolink® (eugenol-free).

#### Mixing and applying:

approx. 30 sec.

#### Time in the mouth:

approx. 1 min. 30 sec.

#### Setting time:

approx. 5–6 min.

#### Ordering Information

##### tempofit® premium 10:1

Standard pack A2 **02201**

Cartridge of 50 ml, 10:1  
10 Mixing cannulas, blue

Standard pack A3 **02209**

Cartridge of 50 ml, 10:1  
10 Mixing cannulas, blue

Standard pack A3,5 **02217**

Cartridge of 50 ml, 10:1  
10 Mixing cannulas, blue

Mixing gun 10:1 **02808**

Mixing cannulas 10:1 **02183**

blue, 25 pcs.

##### tempolink®

tempolink® single **02598**

Cartridge of 5 ml, mini-mix 1:1  
8 Mixing cannulas, brown

tempolink® multi **02596**

Cartridge of 5 ml, mini-mix 1:1  
8 Mixing cannulas, brown

tempolink® clear **02593**

Cartridge of 5 ml, mini-mix 4:1  
8 Mixing cannulas, brown

Mixing cannulas 1:1 **02761**

brown, 25 pcs.

Mixing cannulas 4:1 **02591**

brown, 25 pcs.